

How to use the Prendē Podicure Wax PODS and ORB

<p>Your Orb Box Contains:</p> <ul style="list-style-type: none"> • Orb • 1 x Base Plate With Plug • 2 x Product Cups 			
	Step 1: Unpack ORB and switch on	Step 2: Directly place the POD into ORB	Step 3: Place ORB on plugged in dock
			
Step 4: Put on lid	Step 5: Wait until light turns solid (20-25 min)	Step 6: Add oil to cup	Step 7: Massage Oil onto feet.
			
Step 8: Brush wax onto skin	Step 9: Brush REALLY SLOWLY	Step 10: Cover desired area with wax	Step 11: Cover foot with plastic sheet
			
Step 12: Cover feet with towels/booties	Step 13: Gently massage and loosen	Step 14: Remove all wax	Step 15: Massage in Ureka Footcare Cream

			
Step 9: Brush wax onto skin	Step 10: Brush REALLY SLOWLY	Step 11: Cover desired area with wax	Step 12: Cover foot with plastic sheet
			
Step 13: Cover feet with towels/booties	Step 14: Gently massage and loosen	Step 15: Remove all wax	Step 16: Massage in Ureka Footcare Cream

1. Using more than one product in your ORB takes a little practice but makes your treatments more luxurious, relaxing and professional
2. All products feel better applied SLOWLY
3. All product application feels better when bristles of brush are fully in contact with skin
4. Make sure you warm the brush right through by holding down on the metal plate in cup
5. It takes about 15 minutes for the ORB to heat the skincare product up and less time for your 2nd product cup as the ORB is already warmed
6. Light will begin to flash again after 1 hour to let you know you need to reheat or finish treatment