

HOW TO USE PP20 SILICONE MOULDING PUTTY



1. Measure equal quantities of part PP20-A and part PP20-B.



2. Mix both parts together in the hands for 1-2 minutes.




3. After 1-2 minutes, an even colour is achieved.



4. Form the mixture into the appropriately shaped device and leave in place to set for up to 4 minutes.



5. Ensure the orthotic is completely set before attempting to trim.

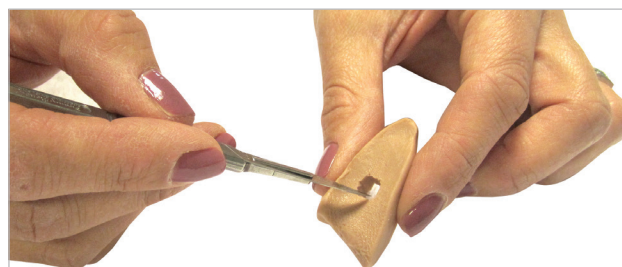

6. If necessary, the orthotic can be trimmed with a scalpel after setting.



7. Final smoothing can be completed using a drill.



8. The final orthotic is very strong and resists tearing, allowing the user to produce more complicated, longer lasting orthotics.